



Urine Adulteration Test Strip




Urine Drug Adulteration Test Strips check for pH, bleach, pyridinium chlorochromate, creatinine, glutaraldehyde, nitrite, and specific gravity. These tests help ensure the integrity of urine samples before drugs-of-abuse testing.

FEATURES

- Detects possible dilution using creatinine and specific gravity
- Detects adulterants such as nitrite, glutaraldehyde, bleach, pyridinium chlorochromate, and other oxidizing agents
- Detects possible contamination by acidic (vinegar) or basic (ammonia solution) substances via the pH test
- Results can be read immediately and up to 45 seconds
- Visual reading by comparing against the color chart
- Supplied in a vial containing 25 strips
- Up to 6 or 7 parameters tested on 1 strip

Test Procedure:

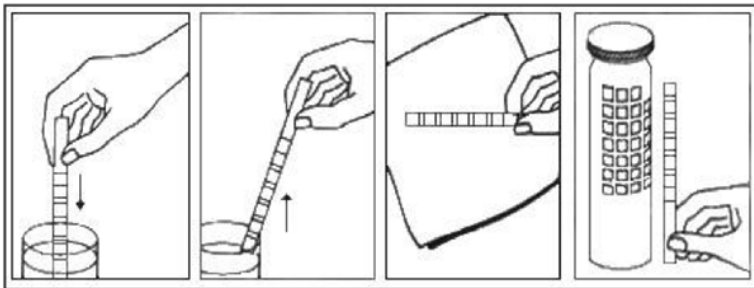

TESTING AND READING TIME / TEMPS DE TEST ET DE LECTURE

Test and Reading Times Temps de test et de lecture	Normal	Abnormal Anormal
pH Immediate / Immédiat	4,0 6,0 7,0 8,0	2,0 3,0 9,0 12,0
Bleach / Eau de Javel 30 seconds / 30 secondes	Negative Négatif	
Pyridinium Chlorochromate / Chlorochromate de pyridinium 30 seconds / 30 secondes	Negative Négatif	
Creatinine / Créatinine 45 seconds / 45 secondes	20mg/dl 100mg/dl	0mg/dl 10mg/dl
Glutaraldehyde / Glutaraldéhyde 45 seconds / 45 secondes	Negative Négatif	50mg/dl 100mg/dl
Nitrite 45 seconds / 45 secondes	Negative Négatif 20mg/dl	50mg/dl 100mg/dl
Specific Gravity / Gravité spécifique 45 seconds / 45 secondes	1,005 1,015 1,025	1,000 ≥1,035

Product Description	Catalog No.	Parameters	Kit Size
Urine adulteration test strips	HUAD-174	7 (pH, bleach, creatinine, glutaraldehyde, nitrite, specific gravity, pyridinium chlorochromate)	25 strips/vial
Urine adulteration test strips	HUAD-164	6 (pH, bleach, creatinine, glutaraldehyde, nitrite, specific gravity)	25 strips/vial

Healgen Scientific, LLC 3818 Fuqua St. Houston, TX 77047, USA www.healgen.com

Healstone Biotech Inc., Unit #650-655 West Kent Avenue North, Vancouver, B.C., Canada, V6P 6T7

1-800-919-3622/1-604-800-2434/ info.healstone@healgen.com www.healstone.com